

IR Channel Settings

The telephone keypad of the GewaTel 200 has been preset to the following Gewa IR transmission channels. Only the **R** key has not been assigned an IR channel.

<u>GewaTel Key</u>	<u>Gewa-Link Channel</u>	<u>PROG Key</u>
1	16	1 / Level 2
2	17	2 / Level 2
3	18	3 / Level 2
4	19	4 / Level 2
5	20	5 / Level 2
6	21	6 / Level 2
7	22	7 / Level 2
8	23	8 / Level 2
9	24	9 / Level 2
0	25	0 / Level 2
*	26	* / Level 2
#	26	# / Level 2
OK	28	A / Level 2
▲ (Volume Up)	29	B / Level 2
▼ (Volume Down) ...	31	D / Level 2
C (Clear)	30	C / Level 2
MENU	42	* / Level 3
Mute	43	# / Level 3
R	(no IR setting)	

See Page 14 for instructions for adding Speed Dial numbers to the phonebook and to assign IR channels to them.

IMPORTANT:

Note the "or" after the instructions to enter a name.

DO NOT PRESS OK, send the IR signal associated with that telephone number to finish.