



Optimist 3 Switch Inputs

Optimist 3 integrates two (2) scan-switch inputs eliminating the need for an external switch adapter. The switch inputs expect a normally open switch. The communication software will recognize when the switch(es) close. No special drivers are required.



Switch 1 (F1)

Switch 2 (F2)

Switch 1 simulates the **F1** key and **Switch 2** simulates the **F2** key.

Here are a few pointers for setting up your software to make use of the switch inputs...

The Grid 2

User Settings/Input Settings/Switch/Connection/Switch Adapter:
Keyboard F1-F8.

- Switch 1 (Primary Switch) = F1
- Switch 2 (Accept) = F2

Connect 1 or 2 switches and press them to test.

BM/SDP

Dynamic Boards/Access Method/Auto Scan/Key Click. Press either Switch 1 or Switch 2 to change the selection to F1 or F2.

VS Communicator

Settings/Input Method/Key-controlled scan/Select Keys: Choose F1 for Key 1 and choose F2 for Key 2.