

Restoring PolyTablet After Low Battery

- 1) Connect AC adapter/charger and external USB keyboard.
- 2) Momentarily press the Reset Button (to the left of this label).
- 3) After Windows appears, there should be an "Unidentified USB Device" input box showing. Do not type anything or close the box. Just proceed to the next step.
- 4) Run the "PERSONA.x.CAB" file located in the "Do Not Delete" folder on the Storage Card. Answer "OK" or "Yes to All" for each step.
- 5) Return to the Windows Desktop where the "Unidentified USB Device" input box is still showing and type: "FTDI_SER.DLL" (without quotation marks). Click OK when done.
- 6) Run Persona by tapping on the "Persona" menu item located in the Start/Programs menu.